









Powered Wheelchair Fauteuil roulant électrique Elektrische rolstoel

000690800.02

Quickie Q500 M/F/R


Instructions for Use Manuel d'utilisation Gebruikershandleiding Wheelchair Components EN

We at SUNRISE MEDICAL have been awarded the ISO-9001 certificate, which affirms the quality of our products at every stage, from R & D to production. This products meet the requirements in accordance with EC guidelines. Options or accessories shown are available at extra cost.

Description du fauteuil FR

SUNRISE MEDICAL est certifié ISO -9001, une norme qui garantit la qualité des produits à toutes les étapes, de la conception à la production, en passant par la recherche et le développement. Ce produit est conforme aux directives de la Communauté européenne. Les options ou accessoires illustrés sont disponibles moyennant un coût supplémentaire.

Rolstoelonderdelen

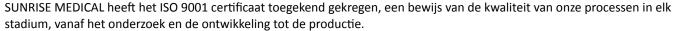

Deze producten voldoen aan de vereisten in overeenstemming met Europese richtlijnen. Getoonde opties en accessoires zijn tegen betaling verkrijgbaar.







If you have any queries about the use, maintenance or safety of your wheelchair, please contact your local approved Sunrise Medical service agent. If you do not know of an approved dealer in your area or have any other questions please write or telephone:

Sunrise Medical Thorns Road Brierley Hill West Midlands DY5 2LD England

Phone: 0845 605 66 88 Fax: 0845 605 66 89 www.SunriseMedical.co.uk

### Dealer signature and stamp

#### **Contents**

1.0 User information	
1.1 This user manual	
1.2 For further information	
1.3 Symbols used in this manual	
2.0 Safety	
2.1 Symbols and labels used on the product	
2.2 Safety: Temperature	
2.3 Safety: Moving parts, (Fig.2.10)	
2.4 Safety: Electromagnetic radiation	
2.5 Safety: Choking hazard	
2.6 Safety: Using a (vehicle mounted) wheelchair lift	
2.7 Safety: Trays	
2.8 Safety: Lifting the wheelchair	
3.0 Intended use of the wheelchair	
3.1 Area of application: The user	
3.2 Area of application: The user environment	
4.0 Setting up the wheelchair	10
4.1 Seating system	
4.2 Seat height	
4.3 Centre of gravity	
4.4 Controller height and depth	
4.5 Control system programme	
5.0 Using the wheelchair	11
5.1 Checking wheelchair before use	11
5.2 Making a transfer	
5.3 Driving the wheelchair	
5.4 Curves	
5.5 Braking & Emergency stop	
5.6 Driving on a slope	
5.7 Obstacles & kerbs:	
5.8 Pushing the wheelchair	
5.9 (Powered) seating options	
6.0 Batteries, charging and range	
6.1 Batteries	
6.2 Charging batteries:	
6.3 The range of your vehicle:	
6.5 Replacing batteries	
6.6 Disconnecting batteries for air transport.	
7.0 Transportation	
7.1 Transport of the wheelchair in a car	
7.2 The wheelchair as a passenger seat	
7.4 General transport warnings	
8.0 Maintenance & cleaning	
8.1 Maintenance	
8.2 Tyre maintenance and pressures	
8.3 Wheels & tyres maintenance	
8.5 Cleaning and disinfection	
8.6 Medium to long term storage:	
9.0 Disposal	
10.0 Trouble shooting	34
11.0 Technical specifications: Applicable norms / standards	2/
12.0 Warranty	41

#### 1.0 User information

Thank you for choosing a Sunrise Medical wheelchair. Sunrise Medical's high-quality mobility products are designed to enhance independence and make your everyday life easier.

As a part of our ongoing product improvement initiative, Sunrise Medical reserves the right to change specifications and design without notice. However, any changes to information provided for existing users shall be clearly communicated if they are safety critical.

Further, not all features and options offered are compatible with all configurations of the wheelchair.

All dimensions are approximate and may be subject to change. The intended lifetime of the wheelchair is 5 years. Please DO NOT use or fit any 3rd party components to the wheelchair unless they are officially approved by Sunrise Medical.

#### 1.1 This user manual

This user manual will help you to use and maintain your wheelchair safely. This entire user manual for this wheelchair consists of three booklets:

- The general wheelchair user manual (this booklet)
- The user manual for the seating system
- The user manual for the controller

When necessary this general user manual refers to other manuals as shown in the table below:

Seating system:	Refers to the user manual for the seating system.
Controller:	Refers to the user manual for the controller.
Battery charger:	Refers to the user manual for the battery charger.

# Do not use your wheelchair until this entire manual and all relevant booklets have been read and understood!

If one of the user manuals was not included with your wheelchair, please contact your dealer immediately.

#### 1.2 For further information

Please contact your local, authorized Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorized dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to **www.SunriseMedical.co.uk** 

#### 1.3 Symbols used in this manual

**⚠** DANGER!

Potential risk of injury or serious injury or death

MARNING!
Potential risk of injury.


**A** CAUTION!

Potential damage to equipment.


#### NOTE:

#### General user advice.

Not following these instructions may result in physical injury, damage to the product or damage to the environment!

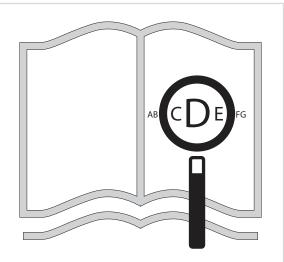

If you are visually impaired, this document can be viewed in PDF format at www.SunriseMedical.co.uk

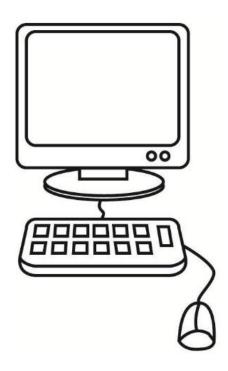

or alternatively is available on request in large text.



#### 2.0 Safety

Follow the instructions carefully next to these warning symbols! Not paying careful attention to these instructions could result in physical injury or damage to the wheelchair or the environment. Wherever possible, safety information is provided in the relevant chapter.

#### 2.1 Symbols and labels used on the product

The signs, symbols and instructions affixed to the wheelchair comprise part of the safety facilities. They must never be covered or removed. They must remain present and clearly legible throughout the entire lifespan of the wheelchair.

Replace or repair all illegible or damaged signs, symbols and instructions immediately. Please contact your dealer for assistance.

- Fig. 2.1. Warning Do Not Touch HOT
- Fig. 2.2. Taxi fixation point
- Fig. 2.3. Warning Danger of finger entrapment
- Fig. 2.4. Warning Ensure that the seat interface quick release is screwed in tight before using your wheelchair, especially on a slope
- Fig. 2.5. Warning Do not drive your wheelchair on a slope with the backrest reclined and/or the seating in standing mode
- Fig. 2.6. Warning Maximum user weight for the wheelchair with Sedeo Pro: 160 kg. With Sedeo Lite: 136 kg
- Fig. 2.7. Freewheel mechanism: outward lever down = freewheel
- Fig. 2.8. The serial number and information label, (example only).
- Fig. 2.9. Location of Serial Number Label, (*overleaf* Fig. 2.9.1 Mid wheel, Fig. 2.9.2 Front Wheel, 2.9.3 Rear Wheel).

#### 2.2 Safety: Temperature

### **M** warning!

- Avoid physical contact with the wheelchair's motors at all times. Motors
  are continuously in motion during use and can reach high temperatures.
  After use, the motors will cool down slowly. Physical contact could cause
  burns. Allow the motors after using at least 30 min. to cool down.
- If you do not use the wheelchair, ensure that it is not exposed to direct sunlight for lengthy periods of time. Certain parts of the wheelchair, such as the seat, the back and the armrests can become hot if they have been exposed to full sunlight for too long. This may cause burns or allergic reactions to the skin.
















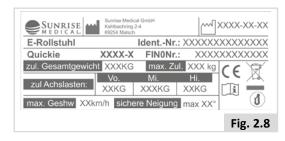

#### 2.3 Safety: Moving parts, (Fig.2.10)



A wheelchair has moving and rotating parts. Contact with moving parts may result in serious physical injury or damage to the wheelchair. Contact with the moving parts of the wheelchair should be avoided.

- Wheels (turning and castor)
- Powered tilt in space adjustment
- Powered seat elevator option
- Powered recline adjustment
- Swing-away bracket of the controller

#### 2.4 Safety: Electromagnetic radiation

# **⚠** CAUTION!

The standard version of your electric wheelchair has been tested on the applicable requirements with respect to electromagnetic radiation (EMC requirements) In spite of these tests:


It cannot be excluded that electromagnetic radiation may have an influence on the wheelchair. For example:


- mobile telephony
- large-scale medical apparatus
- other sources of electromagnetic radiation
- It cannot be excluded that the wheelchair may interfere with electromagnetic fields. For example:
- shop doors
- burglar alarm systems in shops
- garage door openers


In the unlikely event that such problems do occur, we request that you notify your dealer immediately.


### A DANGER!

- When operating two-way radio, walkie-talkies, C.B., Amateur radio, public mobile radio and other powerful transmitting devices the wheelchair should be brought to a halt and turned off.
- The operation of cordless, mobile telephones and cell phones including hands-free devices is permitted but if abnormal operation of the wheelchair is encountered then the chair must be brought immediately to a halt and turned off.

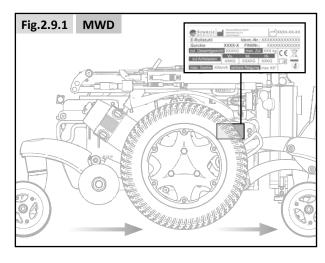

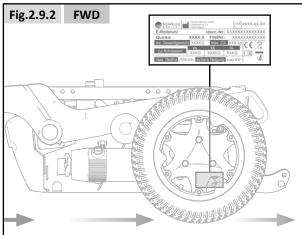

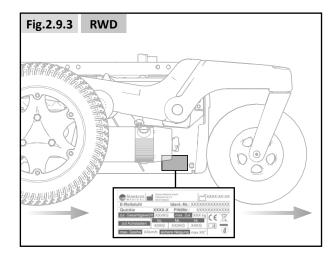

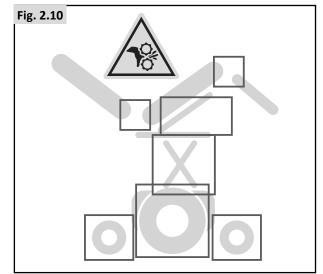

#### 2.5 Safety: Choking hazard



This mobility aid uses small parts which under certain circumstances may present a choking hazard to young children.

#### 2.6 Safety: Using a (vehicle mounted) wheelchair lift

Wheelchair lifts are used in vans, buses and buildings to help you move from one level to another.



- Ensure that the user and all carers fully understand the lift manufacturer's instructions for using the passenger lift.
- Never exceed the lift manufacturer's recommended safe working load and load distribution guidance.
- Always turn off all power when you are on the lift. If you fail to do so, you may touch the joystick by accident and cause your chair to drive off the platform. Be aware that a roll-stop at the end of the platform may not prevent this.
- Always position the user securely in the chair to help avoid falls while on the lift.
- · Always ensure the chair is in drive mode when using passenger lift (wheels locked not in freewheel mode).

#### 2.7 Safety: Trays

Safety: Swing away tray:



- The maximum weight allowed for the tray is 2.5kg.
- Do not overload the tray, this could cause the tray to break or could cause the chair to become unstable.
- Do not leave lit cigarettes or other heat sources on the tray as this could cause the tray to deform and mark.
- Ensure that all extremities and clothing are clear when positioning the tray for use.

#### 2.8 Safety: Lifting the wheelchair



# **⚠** WARNING!

Do not lift this seating system by any parts that are removable, doing so may result in damage to the seating system or injury to the user.

#### 3.0 Intended use of the wheelchair

#### **General description**

The Q500 range is an ultimate modular power wheelchair. A distinctive feature is the suspension system that allows you to have a perfect driving experience indoors as well as outdoors.

Due to its modular design, simplicity and wide range of adjustments, the Quickie Q500 series is a perfect choice for easy service, refurbishment and recycle requirements.

The Q500 series fulfils individual preferences, needs or circumstances and is available in the following configurations:

Midwheeldrive (MWD): Q500 M
Frontwheeldrive (FWD): Q500 F
Rearwheeldrive (RWD): Q500 R

To have optimum seating support and individual adjustments the Q500 series is available in combination with

• Sedeo Lite seating system (refer to seating manual )

• Sedeo Pro seating system (refer to seating manual )

#### 3.1 Area of application: The user

Power wheelchairs are exclusively for a user who is unable to walk or has limited mobility, for their own personal use indoors and outdoors. Driving a powered wheelchair requires cognitive, physical and visual skills. The user must be able to estimate and correct the results of actions when operating the wheelchair.


The wheelchair cannot transport more than 1 person at a time. The maximum weight limit (includes both the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the chassis of the chair (Fig. 2.9).

The variety of fitting variants as well as the modular design mean that this wheelchair can be used by those who cannot walk or have limited mobility e.g. because of:

- Paralysis
- Loss of extremity (leg amputation)
- · Extremity defect deformity
- Joint contractures/joint injuries
- · Strokes and brain injuries
- Neurological disabilities (e.g. MS, Parkinson...)
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia as well as for elderly people who still have the strength in the upper body.
- Persons who are mentally and physically able to control an input device to operate the chair and its functions in a safe way.
- The maximum user weight shall not exceed 160 kg with Sedeo Pro, 136 kg with Sedeo Lite.

#### Basic Components (Fig.3.1).

Headrest **Front Castor** 1. 7. 2. Joystick 8. Drive Wheel 3. Arm Rest 9. **Rear Castor** 4. **Side Cushion** 10. Motors 5. **Seat Cushion** 11. Battery Box 6. Leg Rest 12. Back Upholstery

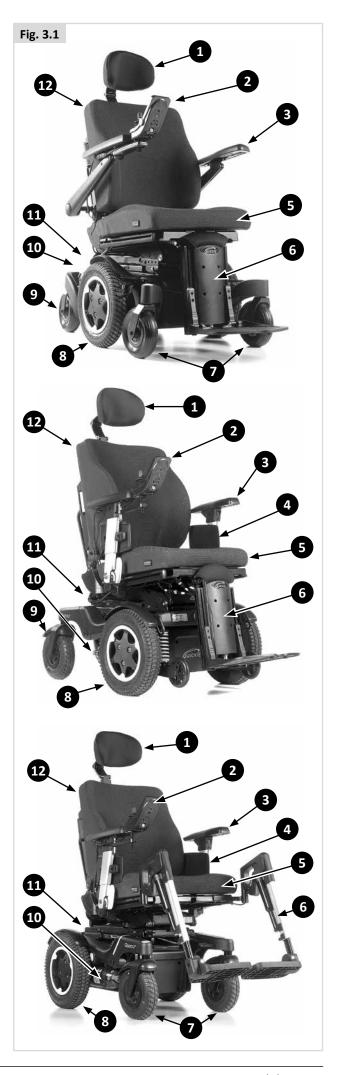

### NARNING!

- If you are under the influence of medicines that can have an effect on your ability to drive, you are not permitted to drive a wheelchair.
- Adequate vision is required in order to safely operate a wheelchair in the user situation concerned.
- Not more than one person at a time can be seated in the wheelchair.
- Do not allow children to ride in the wheelchair unsupervised.

## **A** CAUTION!

• The user of the wheelchair is at all times completely responsible for complying with the applicable local safety regulations and guidelines.

#### **Attendants**

When an Attendant Control Module is fitted, the Power Wheelchair may be operated by an assistant on behalf of the user.

When a Dual Control Module is fitted the Power Wheelchair may be operated by the user, or control may be switched to an assistant to operate on behalf of the user.

- Make sure you heed all warnings and follow all instructions in each section of this manual. Warnings that apply to the user also apply to you.
- Do not stand or sit on any portion of this seating system.
- You need to work with the user, and the user's doctor, nurse or therapist, to develop safe methods best suited to your abilities and those of the user.
- Check to make sure push handle grips will not rotate or slip off.
- Make sure you always have comfortable access to the attendant controls while the chair is moving and make sure that the controller is fixed securely to the chair.
- Set the speed of the attendant control to a speed that you can comfortably follow.
- Always turn off the power to the controller when leaving the user in the chair.

#### 3.2 Area of application: The user environment


This wheelchair has been designed to accommodate both indoor and outdoor use (EN12184 (2014) Class B). When driving the wheelchair outdoors, drive only on paved roads, pavements, footpaths and bicycle paths. The speed must be adapted to suit the environment.

# **⚠** WARNING!

- Drive carefully on slippery roads resulting from rain, ice or snow!
- You are required to turn on the lights in case of limited visibility.
- When driving at higher speeds you must be extra careful.
   Select a lower maximum speed indoors, on the pavement and in pedestrian areas.
- Do not drive off high obstacles.
- Do not attach a weight to the wheelchair without the approval of a qualified specialist. This may negatively affect the stability of the product.

# **A** CAUTION!

- Prevent the wheelchair from coming into contact with sea water: sea water is caustic and may damage the wheelchair.
- Prevent the wheelchair from coming into contact with sand: sand can permeate into the moving parts of the wheelchair, causing extensive wear on these parts.
- Do not use the wheelchair if temperatures are below: -25°C or above +50°C.
- Do not open doors using the leg rests.
- Do not push and/or tow any objects with the wheelchair.
- Do not drive through puddles of water.

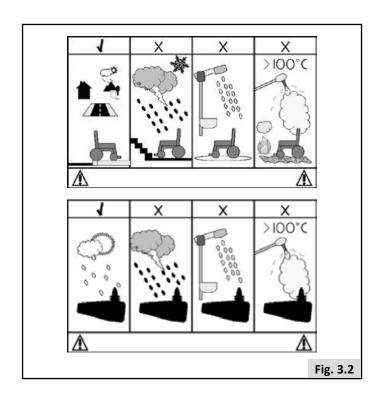

#### 4.0 Setting up the wheelchair

Sunrise Medical wheelchairs can be adjusted according to specific needs of the user.

In this, we make a distinction between wheelchair settings and wheelchair seating adjustment options. Settings are carried out once, the first time the wheelchair is going to be used. They must be executed by qualified specialist, unless explicitly stated otherwise. Seating adjustment options can be carried out by the user. These adjustment options do not require the use of tools.

#### 4.1 Seating system

The extensive settings with respect to the seating system allow for optimum support of the body ( Seating system). The following setting options are possible:

- 1. Seat depth and seat width
- 2. Backrest angle
- 3. Armrest height/width
- 4. Lower leg length
- 5. Angle of the footplate
- 6. Legrest angle

#### 4.2 Seat height

Depending on the specific circumstances of the user of the wheelchair the seat height can be set. Every optional seat height lies within the maximum safe slope as described in EN12184 (2014) Class B. Lowering the seat height has a positive influence on the (dynamic) stability.



Increasing the seat height has a negative effect on dynamic stability.

#### 4.3 Centre of gravity



Adjusting the centre of gravity on a powered wheelchair is both physically demanding and safety critical. Sunrise Medical recommends that this adjustment is only performed by a service agent/dealer approved by Sunrise Medical.

#### 4.4 Controller height and depth

( Seating system).

#### 4.5 Control system programme

This wheelchair is equipped with a control system that can be programmed by adjusting settings within the system. This is a specialized job that needs training and can only be adjusted with professional software. The end user is not allowed to adjust these settings.



The incorrect setting of the parameters for the control system may result in very dangerous situations.

Settings must be made by qualified specialists.

#### 5.0 Using the wheelchair



- Be aware that you may need to adjust the controller settings of your system.
- Consult your authorised dealer to adjust the control settings immediately if you notice any change in your ability to:
- 1. Control the joystick
- 2. Hold your torso erect
- 3. Avoid running into objects.

#### 5.1 Checking wheelchair before use

Perform the following daily check routine before driving:



#### Checking wheels & tyres

• Are the tyres sufficiently inflated (Chapter 8.2).

#### **Checking batteries**

- Before using your vehicle for the very first time, please charge your batteries for a period of 24 hours.

#### **Checking lights and indicators**

 Ensure that the lights and indicators are functioning correctly and lens are clean before going outdoors at night.
 Controller

#### **Checking remote**

• With the control system switched off, check that the joystick is not bent or damaged and that it returns to the centre when you push and release it.

#### Checking free wheel lever

 Ensure that the free wheel lever has been set to 'drive'. (Chapter 5.8).

#### **Checking seating**

- Ensure that all the cushions are in place.
- Make sure that the backrest is correctly fitted and adjusted.
- Visually inspect the wheelchair to make sure the leg rests, armrests etc. are correctly positioned and attached to the wheelchair and all fasteners are sufficiently tightened.
- Ensure that the seat interface knobs are tight and secure.

#### Checking clothing on potential entrapment

 When operating the wheelchair, ensure that your clothing does not hamper the wheelchair (i.e. too long). Before use, always check if your clothing or accessories do not come into contact with the wheels or and other moving and/or rotating parts in which they could become entangled.

#### **Checking weather conditions**

• In winter, batteries have a reduced capacity. During a period of light frost, the capacity is roughly 75% of the normal capacity. At temperatures below -5°C this will be roughly 50%. This will reduce your range of action.

#### 5.2 Making a transfer

Sunrise Medical recommend that you consult your healthcare professional for assistance in developing your personal front or side transfer technique to best suit your needs and avoid any personal injury.

#### Preparing for a forward transfer:



It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the seating system seat is not below you.

For further instructions and warnings about making a transfer, please refer to  $\square$  Seating system.

#### 5.3 Driving the wheelchair



#### ✓!\ WARNING

• The user of the wheelchair is at all times completely responsible for complying with the applicable local safety regulations and guidelines.

Powered wheelchairs are driven by means of a controller. There is a separate user manual for the controller and this was included with this wheelchair ( Controller).

- 1. Switch on the controller.
- 2. Set the maximum speed limit.
- 3. Move the joystick in the direction in which you would like to go.
- 4. Moving the joystick even further forwards will cause the wheelchair to drive faster.

#### Rear view mirror



- To avoid injury to people around you please be aware that the mirror protrudes outside the space envelope of the chair and could cause injury to someone when driving past.
- The mirror must be used on models at speeds over 6 kph on roads according to local law.
- Always make sure that when using the mirror that it is clean and unbroken so that it does not impair your visibility.

#### Road use

Please show the utmost consideration for the other traffic on the road.



- Remember that the last thing a car or lorry driver expects to see is a wheelchair backing off the kerb into the road.
- If in any doubt, do not risk crossing the road until you are certain that it is safe.
- Always cross the road as quickly as possible, there may be other traffic.
- Do not drive over anything that could cause punctures in the tyres.
- Ensure that there are no objects in your path that could possibly become lodged in your chair mechanism or in the spokes of the rear wheels. This could cause the chair to come to a sudden stop.
- Riding over drains or grids could cause the wheelchair castors or wheels to become lodged, causing the chair to come to a sudden stop.

#### Adverse conditions:

Please be aware that when driving your wheelchair in adverse conditions, e.g. on wet grass, mud, ice, snow or other slippery surfaces, you may experience a reduction in the grip and traction of your wheelchair.



- We recommend you take extra precautions in these conditions, particularly on hills and slopes; your wheelchair could become unstable or skid causing possible injury.
- When you are using a Powered Wheelchair take extra care with loose or long items of clothing. Moving parts, such as wheels, can be potentially dangerous or even fatal if clothing becomes entangled.

#### NOTE:

Extreme variances in temperature may trigger the self protect mechanism in the control system. If this occurs the control system will temporarily shut down to prevent damage to the electronics or the wheelchair.

#### 5.4 Curves



Full speed turns should not be attempted. If you need to turn sharply you must reduce your speed with the joystick or speed setting. This is particularly important when travelling across or down a slope. Disregarding this advice could lead to your wheelchair tipping over.



Use indicators when changing direction.

#### 5.5 Braking & Emergency stop

There are three ways to stop your wheelchair:

- Simplest and safest way to stop the wheelchair is to release the joystick. This will bring the chair to a halt in a controlled manner
- Pulling back the joystick will break the chair abruptly with a fast stop.
- Switching the control system off whilst the chair is in motion will also bring the chair to a halt.

# **⚠** WARNING!

- Switching the control system off is only to be used in an emergency situation as the stopping action is very abrupt.
- Never try to stop or slow this seating system with the wheel locks (if applicable). Wheel locks are designed to pre-vent the system from moving when stopped. They are not brakes.

#### 5.6 Driving on a slope

Your wheelchair has been designed and tested to allow its use on slopes or gradients of up to:

Q500 M: 8° (14%) in standard configuration. Q500 F: 8° (14%) in standard configuration. Q500 R: 10° (17.6%) in standard configuration.

However, you have the option of adjusting your position with the seat elevator, tilt in space, powered recline or powered centre mount leg rest or a combination of these options.



- Elevating, tilting and leg rest adjustments can affect the
  wheelchair centre of gravity. Under certain conditions this
  might cause instability that could lead to unintentional
  tipping. It may be necessary to carry out a risk assessment if
  the wheelchair is adjusted to extremes.
- Stopping distances on slopes can be significantly longer than on level ground.



- In certain circumstances your wheelchair could become unstable.
- Before attempting to climb or descend a slope or a kerb, caution should be taken when using weight shift options (e.g. powered, recline or standing), of the seat and/or your body for a counter balance weight.
- To improve stability lean forward when driving uphill, with the seat and back in an upright position.
- Alternatively sit in an upright position when travelling in a forward, downhill direction or and/or recline the seat backwards.
- We strongly recommend that you return the seat and back to their home positions before attempting to climb or descend a slope. Failure to do this may cause the wheelchair to become unstable.
- If you are in any doubt about the capabilities of your wheelchair on a slope then do not attempt to drive up or down the slope/kerb; try to find an alternative route.

#### **Gradients: ascents:**



- · When going uphill, keep the chair moving.
- Steer by carefully moving the joystick forwards making slight Left and Right adjustments as you go.
- If you have stopped on a hill, you should start slowly.
- If necessary lean forward.

#### **Gradients: descents:**

On descents, it is important not to let the wheelchair accelerate beyond its normal level of ground speed.



- Proceed slowly down steep descents, (below the speed of 5kph) and stop if you fell anxious.
- If the chair picks up speed, centre the joystick to its home position to allow it to slow, or to stop.
- Restart slowly and do not allow the speed to increase above what you are comfortable with.

#### NOTE:

 The controller has the benefit of a logic system that will help compensate when driving along a camber or up a hill. This is an added safety feature on your wheelchair. In addition of course, you may control the wheelchair speed by using the speed control.

#### 5.7 Obstacles & kerbs:



- Never descend a kerb backwards.
- Do not attempt to climb or descend a series of steps or use on escalators. It is unsafe to do so and could cause personal injury or damage the chair. This wheelchair has only been designed to climb a single step or kerb.
- We recommend that users with upper trunk instability wear restraint systems to keep the upright body position during descending or ascending ramps, kerbs or obstacles.

#### Kerb climbing:

Always approach a kerb at 90° (Fig. 5.5).

- Approach the kerb or step, head on at a 90° angle.
- · Drive forwards slowly and steadily.
- Stop the chair as soon as the castor wheels touch the kerb.
- Apply sufficient power to the motors to lift the front of the chair up onto the kerb or step and then apply slightly more power until the drive wheels climb the kerb or step smoothly.
- As far as possible, keep the joystick in the straight forward position.

The maximum obstacle or kerb climbing height is:

Q500 M: 100 mm.
 Q500 F: 70 mm.
 Q500 R: 80 mm.
 Q500 R + kerbclimber: 100 mm.

The approach speed and process can vary depending on your wheelchair drive type and castor wheel choice.

#### Dismounting the kerb.




Move the chair slowly and carefully in a forward direction until both front wheels are on the edge of the kerb, again in a 90° position to the kerb. (Fig. 5.3).

Drive as slowly as possible off the kerb with the drive wheels. Don't stop the chair during decent of the kerb. You will feel more secure if you can lean backwards, but if you can't, don't worry, the wheelchair is stable. As long as you stay within its limitation, you will be quite safe.

All powered seating options need to be in home position. Your powered leg rests may need to be adjusted to give enough clearance to mount or dismount the kerb.

We recommend to use the lap strap to feel more secure during declining the kerb. For extra protection we recommend to fit every chair with the leg rests.

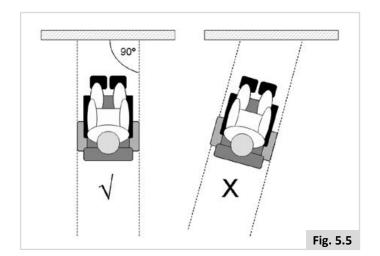

#### 5.8 Pushing the wheelchair

The wheelchair can be moved by pushing. In order to do so, the motors must be disengaged using a freewheel lever. This feature was developed for the attendants of wheelchair users and also functions as an emergency freewheel lever.

The freewheel lever has 2 positions

1. For driving the wheelchair. (Fig. 5.6).

'Drive' position: Turn the lever so that the lever at the drive wheel side is pointing upwards.


2. For disengaging the motors. (Fig. 5.7 - Fig.5.8).


'Push' position: Turn the lever so that the lever at the drive wheel side is pointing downwards.

### A DANGER!

- The free wheel lever should only be operated by the attendant and never by the user.
- A user may never be left unattended while the wheelchair is in 'push'.
- Never set the lever to the 'push' position on a slope! When the freewheel lever is set to 'push', the automatic parking brake is deactivated. This makes it possible for the wheelchair to roll down the slope.
- The automatic parking brake only works is the lever is set to 'drive' position.
- When the wheelchair is no longer being pushed, the freewheel lever should be set to 'drive' immediately.
- To manually push the system you must release the motor brakes.
- Do not engage or disengage motor brakes unless power to the system is off.
- Make sure you have full control over the system when you release the motor brakes. When you do so the system will not have brakes.
- Make sure the system is on level ground before you release the motor brakes.
- Propel this system by the push handles only. They provide secure points for you to hold the rear of the system to prevent a fall or tip-over.
- Check to make sure push handle grips will not rotate or slip off.



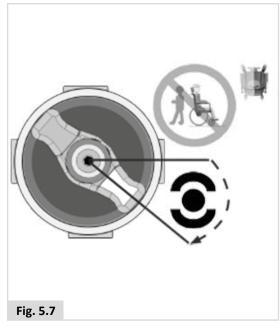

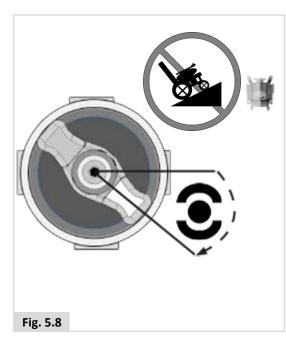

#### 5.9 (Powered) seating options

The wheelchair can be equipped by the following mechanical and/or powered options to adjust your seating position. These options can be operated by the user without the use of tools.


Powered adjustable seating options (Fig.5.9 - Fig.5.11).

How to operate these powered seating options can be found in  $\square$  Controller +  $\square$  Seating:


- 1. Powered tilt function
- 2. Powered seat elevator
- 3. Powered backrest recline
- 4. Powered elevating leg rests

# A DANGER!

- Use of the powered adjustable seating options may influence the stability of the wheelchair.
- Only use the powered adjustable seating options when the wheelchair is on a flat surface! Do Not attempt to operate any power seating option while positioned on an incline.
- Please stay clear of any powered actuator while the component is in motion. Sunrise power mechanisms move over large ranges of travel. Users should be aware of their surroundings while components are in motion.
- While driving this seating system in the reclined, elevated, or tilted position use caution.
- Do Not attempt to operate any power seating option while this seating system is in motion.
- Do not attempt to operate the lift or tilt power seating options around children.
- Through programming it is possible to reverse the direction of all powered seating functions, ensure you know which direction your seat is going to move before operating.
- Please be aware that the lift/tilt modules present a trap hazard.
   Make sure that during operation the lift/tilt modules are free from all clothing, hands, feet and other extremities, to prevent injury.
- Do not drive on ramps or slopes with the seating in the elevated position. Before attempting to climb or descend a slope, return the wheelchair to a standard seated drive position, with the backrest upright.
- This power system is designed to automatically reduce the maximum driving speed to a creep mode or stop when certain power seating limits are reached. Depending on the configuration of the chair there might be multiple creep speeds.

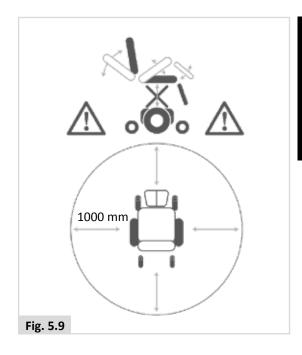



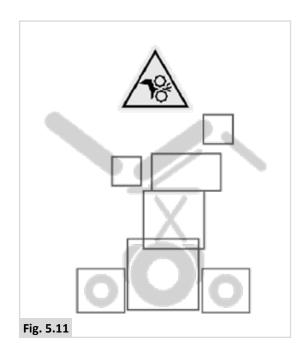

#### 6.0 Batteries, charging and range

#### 6.1 Batteries

Batteries are the power source for almost all of the modern mobility products available today. The design of batteries used in mobility products is significantly different to the batteries used to start a car for example. Car batteries are designed to release a large amount of power over a short period of time, whilst mobility batteries (commonly called deep cycle batteries) release their power evenly, over a long period of time. Therefore, due to the lower production volumes and increased technological requirements, mobility batteries are typically more expensive.


Commonly two 12 volt batteries are used together in a mobility product, giving a total voltage of 24 volts. The size of the battery (e.g. its available power) is expressed in amps per hour e.g. 70amp/hr. The higher the number, the bigger the battery size, weight and, potentially, the greater the distance you can travel. Sunrise Medical only fit as standard maintenance free batteries into these types of wheelchairs. This type of battery uses a method of carrying the electrolyte commonly referred to as 'gel', that is held within the battery case. As the name implies, no maintenance is required other than regular charging. You can safely transport this type of battery without fear of acid spilling. Furthermore, they are approved for transportation on aircraft, trains and ships.



- Do not fit car batteries to the wheelchair. Fit only deep cycle, maintenance free mobility batteries.
- When the batteries are worn out, take them to the local authority disposal point.
- Do not expose any part of the battery to direct heat (i.e. naked flame, gas fire).
- Be extra cautious to reduce the risk of dropping a metal tool
  onto the battery. It could spark or short circuit the battery
  or other electrical parts that may cause an explosion. Also
  take off all personal metal effects and dangling objects when
  working on the battery.
- Return the batteries back to Sunrise Medical or directly to the battery manufacturer for recycling, when they no longer hold charge.
- The battery terminals need to be checked regularly for signs of corrosion. If any corrosion is apparent, then clean the terminals completely and re-grease the terminal using Vaseline petroleum jelly, not ordinary grease. Ensure that the terminal nut and bolt, cable clip and exposed cable are completely covered with jelly.

# A DANGER!

- Do not expose any part of the battery to direct heat (i.e. naked flame, gas fire).
- Be extra cautious to reduce the risk of dropping a metal tool
  onto the battery. It could spark or short circuit the battery
  or other electrical parts that may cause an explosion. Also
  take off all personal metal effects and dangling objects
  when working on the battery.

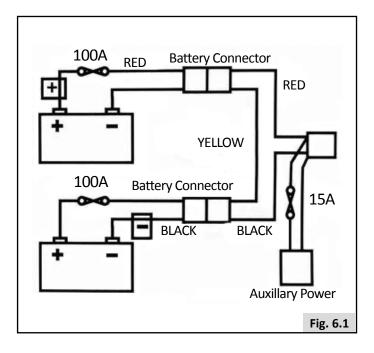

## **⚠** DANGER!

- Do not, under any circumstances, tamper with the batteries.
   If in any doubt contact your local Sunrise Medical authorised dealer.
- Avoid contact with acid on damaged sealed type batteries or wet batteries.
- Battery acid can cause burns to the skin as well as damage to floors, furniture and your wheelchair.
- If battery acid comes into contact with the skin or clothing, wash immediately with soap and water.
- If battery acid comes into contact with the eye, immediately flood the eye with running cold water for at least 10 minutes and seek medical attention immediately.
- Acid spills can be neutralised with baking soda and water.
- Take care to keep batteries upright at all times, especially when transporting your wheelchair.

#### Safety cut-outs

In the event of a short circuit there are several safety systems built into your wheelchair to safeguard your electrical circuits, (Fig. 6.1).

- 1. Fusible 100A links are connected into the battery harnesses to protect the batteries and wiring.
- 2. 15A auxiliary power circuit fuses for auxiliary modules and seating power supply.

To replace them contact your Sunrise Medical authorised dealer, who will also diagnose the original fault.

#### 6.2 Charging batteries:

Battery charger: Please read the owner's manual with the charger supplied carefully. The general procedures and effects for the interference with the chair and the batteries remain valid.

#### **Battery care plan**

Below is set out a battery care plan for maintenance free batteries. This has been agreed between Sunrise Medical and the battery manufacturers, to enable you to get the best out of your batteries. If a different care plan is followed, this may result in lower than expected performance from your mobility vehicle.

- Only use an approved Sunrise Medical charger compatible with the vehicle to be charged.
- Charge your batteries every night, regardless of the amount of use your mobility device has had during the day.
- Do not interrupt the charging cycle.
- If your mobility device is not required for use, it should remain connected to the charger until required. This will not damage your batteries, as long as the mains socket/plug is left switched on. Turning the mains socket/plug off, but leaving the mains cable plugged in will eventually deplete your battery charge.
- If you leave your vehicle for an extended period (more than 15 days) charge the batteries fully and then disconnect the main battery lead.
- Failure to allow for recharge will damage the batteries and can lead to shortened distances and premature failure.
- Do not top up the charge of your batteries during the day. Wait until the evening for a full overnight charge.
- As a general rule, maintenance free batteries take longer to fully charge than "wet" lead acid batteries.
- The battery terminals need to be checked regularly for signs of corrosion. If any corrosion is apparent, then clean the terminals completely and re-grease the terminal using Vaseline petroleum jelly, not ordinary grease. Ensure that the terminal nut and bolt, cable clip and exposed cable are completely covered with jelly.
- Following all the points above should result in a healthier battery, greater range for the vehicle user and a longer life for your batteries.

#### Connecting the battery charger

Procedure for connecting the charger and charging

- Connect the battery charger round output plug to the charging socket on the front of the joystick module.
   Controller:
- Connect the charger to the mains supply by means of the mains plug and switch on.

For more information about the charger operation, consult the instructions provided with the charger. 

BATTERY CHARGER. Depending on how depleted the batteries are, it may take up to 12 hours to fully charge the batteries again.

# ⚠ DANGER!

- Make sure the cord is located so that it will not be stepped on, tripped over or otherwise subjected to damage or stress.
- Do not rest a battery on top of the charger.
- Do not stand the charger on a carpet or other soft surface. Always place it on a hard flat surface.
- Do not operate the charger if it has received a sharp blow, been dropped or otherwise damaged in any way. Take it to a qualified technician.
- Never place the charger directly above the battery being charged; gases from the battery will corrode and damage the charger.
- Never charge a frozen battery. A fully charged battery will
  rarely freeze but the electrolyte of a discharged battery can
  freeze at -9° Centigrade. Any battery that is suspected of
  being frozen should be thawed completely before charging.
- Never sit with the charger on your lap when charging your batteries
- The charger casing will get hot during its normal operation.
- Do not disassemble charger; only have it repaired by the manufacturers. Incorrect re-assembly may result in a risk of electric shock or fire.
- To reduce the risk of an electric shock, unplug the charger from the outlet before attempting any maintenance or cleaning. Turning off the controls will not reduce the risk.
- Never smoke or allow a spark or flame in the vicinity of battery or charger.
- You should not charge your batteries in outdoor conditions.
- It is advised to use a smoke detector in the charging area.

#### **Battery Charger**

The external charger has been designed to charge two 12V Gel type batteries connected in series (= 24 V).

The chargers have features which prevent hazards or accidents occurring as a result of connecting batteries the wrong way round, overheating caused by fault conditions or attempting to charge wrong voltage batteries.

The majority of charger sizes are electrically double insulated and no earth connection is required. Some larger sizes may be electrically earthed and this will be clearly stated on the label.

If your charger has been specified for use in Continental Europe it will contain a European two pin plug which does not have a fuse. In this case the fuse is located in the fascia panel of the charger.

#### Country specific information: UK

The 3 pin UK mains input plug contains a replaceable fuse. The rating of this fuse is shown on the charger label.



- As with all mains powered electrical equipment, always replace blown fuses with the same type and size of fuse as specified.
- Fitting of different fuses can result in an increased fire risk, damage to the charger or failure of the charger to operate properly.

#### 6.3 The range of your vehicle:

Please refer to the specification tables at the back of this manual for Energy Consumption, (Maximum Range), information.

Most manufacturers of mobility products state the range of their vehicles either in the sales literature or within the Owner's Manual. The range stated sometimes differs from manufacturer to manufacturer even though the battery size is the same. Sunrise Medical measure the range of their vehicles in a consistent and uniform manner, but variances still occur due to motor efficiencies and overall product load weight.

The range figures are calculated to I.S.O. Standard 7176. Part 4: Wheelchair Energy Consumption Theoretical Range.

This test is carried out in controlled conditions with new, fully charged batteries, on a level test surface and a user weight of 100 kg. The range figures stated should be seen as a theoretical maximum and could be reduced if any single, or combination, of the following circumstances occur:

- · User weight heavier than 100 kg.
- Batteries whose age and condition are less than perfect.
- The terrain is difficult e.g. very hilly, sloping, muddy ground, gravel, grass, snow and ice.
- The vehicle climbs kerbs regularly.
- The ambient temperature is very hot or very cold.
- Incorrect tyre pressures in one or more tyres.
- · Lots of start/stop driving.
- Also thick pile carpets within the home can affect range.
- Use of additional power consumption options (e.g. light, actuators, etc.)

The battery sizes available on each Sunrise Medical product should give sufficient range to cope with the majority of customer's lifestyles.

#### 6.4 Battery warranty:

Battery warranties are subject to periods set by the manufacturers. However, most of these warranties are subject to a wear and tear clause, and if you genuinely wear out your batteries in 6 months, it will not be possible to obtain a replacement under warranty.

#### 6.5 Replacing batteries

The wheelchair uses two 12V non-spillable 'dry' gel batteries in series, these batteries are fully sealed and require no maintenance.

Replacing and servicing batteries is to be done by qualified specialist.

In case of malfunctioning batteries, contact your local dealer.



• Do not attempt to replace or service batteries without the supervision of trained and qualified personnel.


#### 6.6 Disconnecting batteries for air transport.

The batteries are contained within the drive unit located under the battery shroud.


#### To disconnect the batteries on Mid-wheel drive:


- Release the two handle screws under the front of the seat frame, that connect the seat frame with the seat module interface, (Fig.6.2).
- Flip the seat frame backwards (Fig.6.3).
- Ensure the seat is secured with the seat stay in place, before letting go, (Fig.6.4 A).
- Lift off the battery box lid, (Fig. 6.4 B).
- Disconnect the 2 pin Anderson connectors from each battery (Fig. 6.5).

To reconnect the batteries reverse the above procedure.

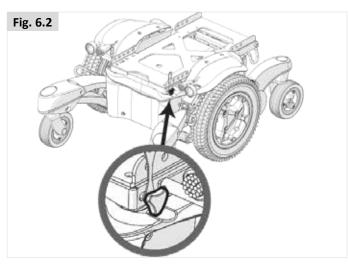



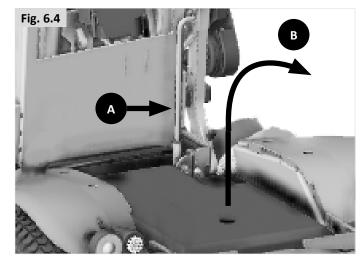

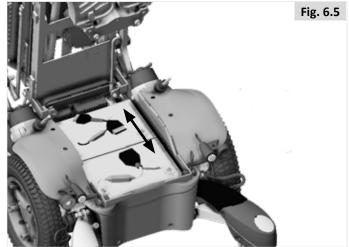

#### To disconnect the batteries on Front and Rear-wheel drive:


- Release the two handle screws under the mid of the seat frame, that connect the seat frame with the seat module interface (step 1) and release the lever (step 2) (Fig.6.6.1 - 6.6.2).
- Flip the seat frame backwards (Fig.6.3).
- Ensure the seat is secured with the seat stay in place, before letting go, (Fig.6.7 Front Fig.6.8 Rear).
- Lift off the battery box lid, (Fig. 6.4 B).
- Disconnect the 2 pin Anderson connectors from each battery (Fig.6.5).


To reconnect the batteries reverse the above procedure.


# MARNING:


Removing the batteries can lead to a change of the center point of gravity. The result may be that the chair flips backwards without the weight of the batteries. Sunrise advises to:

- Putting an obstacle behind the chair to secure the situation. This can be a chest, or a chair, or etc.
- Try to lift the battery slightly first to check the effect of taking out the weight.

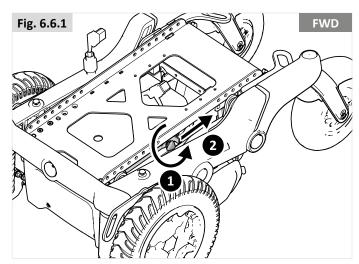

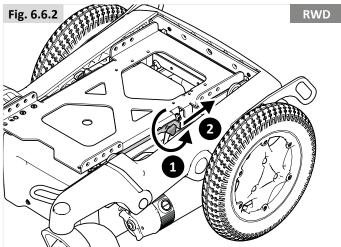

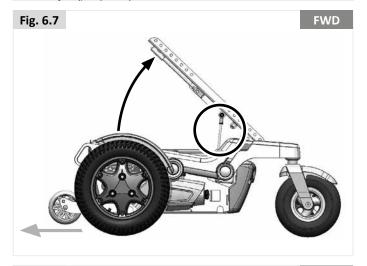

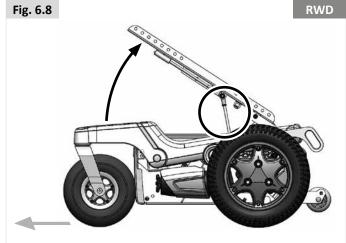

### 7.0 Transportation

#### 7.1 Transport of the wheelchair in a car

#### Getting in and out of the vehicle

### ✓!\ DANGER!

- Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/ egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the occupant, the wheelchair and accessories.
- Use a ramp (or lifting device) suitable for the entire weight of the wheelchair (including user) for entering the vehicle.
- The wheelchair should be secured in a forward facing direction. This wheelchair is approved for use in cars and meets the requirements for forward facing transport and head on collisions. The wheelchair has not been tested for other positions in a vehicle (Fig. 7.1).



- Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
- Ease of access to, and manoeuvrability in, motor vehicles can be significantly affected by wheelchair size and turning radius; smaller wheelchairs and/ or wheelchairs with a shorter turning radius will generally provide greater ease of vehicle access and manoeuvrability to a forward-facing position.

#### Securing the wheelchair with a strap restraint system


This wheelchair has been successfully crash tested in accordance to: ISO 7176-19:2008, using a representing restraint system.

- Secured with a 6-point strap restraint system (2 in the front, 4 in the back, Fig.7.2), that conforms to:
  - i. ISO 10542 (international) or
  - ii. SAE J2249 (USA).

**NOTE:** Wheelchair secured with a 6-point strap restraint system; suitable for the total weight of the wheelchair (including any options) in accordance with the WTORS1) manufacturers instruction.

#### Terminology explained:


1) WTORS: Wheelchair Tie-down Occupant Restraint System

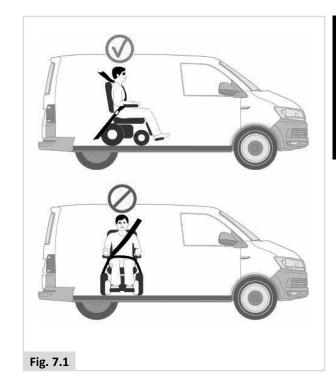

# ⚠ DANGER!

- The wheelchair can only be secured by using the tie-down points on the wheelchair's frame (Fig. 7.3).
- The tie-down points (2 in the front, 2 in the back) are indicated by the tie-down symbol (Fig. 7.2). Fit the front straps first, then the rear. Tension the straps to secure the wheelchair. The wheelchair may not be secured onto any accessories (wishbones, armrests, anti-tip brackets etc.).
- No changes or replacements must be made to the anchorage points/car fastenings on the wheelchair, or to constructional elements or parts of the frame without consulting the manufacturer.
- When the wheelchair is transported in a vehicle without occupant and no restraint system is available then the wheelchair should be properly stored and secured in the cargo area during travel.



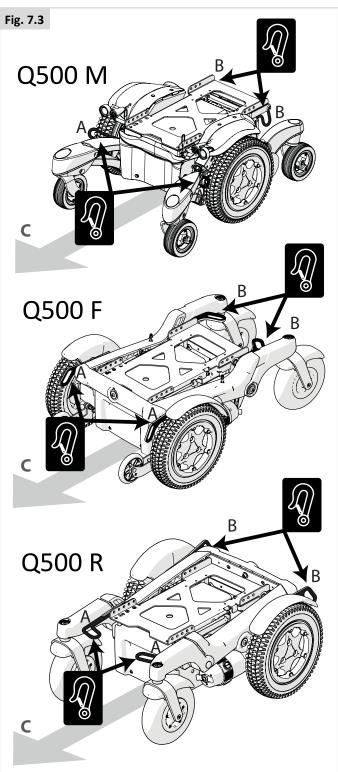

- A Front Tie Down Points.
- B Rear Tie Down Points.
- C Direction of Travel.

#### 7.2 The wheelchair as a passenger seat

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security as a vehicle seating system. Sunrise Medical recommends that the user transfers to the vehicle seating and uses the vehicle-installed restraint system wherever possible. Sunrise Medical recognizes that it is not always practical for the user to be transferred and in these circumstances, where the user must be transported whilst in the wheelchair, the following advice must be followed:

# ⚠ DANGER!

- The safety of the user during transportation depends upon the diligence of the person securing the tie down restraints and they should have received appropriate instructions and/ or training in their use.
- Wherever possible remove and stow safely away from the wheelchair all auxiliary equipment, e.g.: crutches, loose cushions, tray tables, etc.
  - i. Removed and secured separately in the vehicle, or
  - ii. Secured to the wheelchair, but with energy-absorbing padding placed between the table and the user.
- Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported.
- If applicable: Reclining backrests should be returned to an upright position.
- If applicable: Seat elevators should be in the lowest position.
- If applicable: The manual brakes must be firmly applied.

#### **Occupant restraint Instruction**

# **^**

#### DANGER!

- Use a 3-point occupant restraint system to secure the occupant.
- Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components.
- Restraints should be mounted to the appropriate vehicle pillar and should not be held away from the body by wheelchair components such as armrest or wheels (Fig. 7.4).
- Use a suitable positioned headrest when being transported in a wheelchair.
- Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO 7176-19:2008 or SAE J2249.

#### Positioning the occupant restraint



- The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal.
- A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75° degrees (Fig. 7.5).
  - i. The upper torso restraint belt must fit over the shoulder and across the chest as illustrated (Fig. 7.5). Restraint belts must be adjusted as tightly as possible consistent with user comfort.
- Restraint belt webbing must not be twisted when in use.
- Care should be taken when applying the occupant restraint to position the seatbelt buckle so that the release button will not be contacted by wheelchair components during a crash.

#### Passenger with a user weight < 22 kg.

When the user being transported is a child, less than 22 kg mass and the vehicle involved has less than eight (8) seated passengers it is recommended that they be transferred to an UNCE Regulation 44 compliant child restraint system (CRS). This type of restraint system provides a more effective occupant restraint system than the conventional 3 point occupant restraint system and some CRS systems also include additional postural supports to assist in maintaining the position of the child when seated.

Parents or care providers may consider the option, in some circumstances, for their child is to remain in their wheelchair whilst in transport due to the level of posture control and comfort provided by the set up in the wheelchair. We would recommend in such circumstances that a risk assessment be carried out by your healthcare professional and relevant competent persons.





#### 7.3 Special transport requirements

#### Using your wheelchair on the train.

Before you travel, please contact the train operator. They will be able to supply details of any special requirements/instructions. We advise you to check the following:

- Is there a suitable and intended "wheelchair" area in the train for wheelchair users (with adequate turning space to manoeuvre)?
- Is there a suitable or designated area on the platform for wheelchair users (with adequate turning space to manoeuvre)?
- Is there suitable boarding access available to allow you to access the railway carriage and the intended "wheelchair" area?
- Is the boarding access suitable for the combined mass of the wheelchair and the occupant?
- Make sure the slope of the boarding access is not greater than the dynamic safe slope. (See Section 10)
- Any obstacles or thresholds should not exceed the max kerb climbing capability of your wheelchair. (See Section 10)

Most train operators will provide assistance provided that arrangements have been made in advance. We suggest you have you Owner's Manual ready when planning your journey and contacted them.

# ⚠ WARNING!

While driving on a boat or train, make sure that you use a driving profile without GYRO activated if installed as an option to the wheelchair. The continuous movement of a boat or train, disrupts the GYRO system and the driving characteristics of your wheelchair.

#### Transporting the wheelchair as luggage.

Parts of the wheelchair that can easily be detached should be removed when transporting the wheelchair. Store these parts securely. E.g.:

- Remove the seat ( Seating system).
- Remove the leg rests ( Seating system).
- Remove the armrests ( Seating system).
- D. Fold down the backrest ( Seating system).

- Ensure that any detachable parts are secured with your mobility aid or separately packed and labelled so they do not get lost during loading and unloading.
- The wheelchair may be transported by road, rail, sea or air and the batteries conform to IATA regulations.
- Before you travel, please contact the appropriate carrier. The travel operator will be able to supply details of any special requirements/instructions.
- For information about dimensions and weight of the wheelchair, see (Section 10).
- For information about the batteries in your wheelchair see (Section 6).
- Ensure that any detachable parts are secured with your mobility aid or separately packed and labelled so they do not get lost during loading and unloading.
- Take this user manual, the seating system user manual (& Seating System) and the controller user manual (& Controller) with you. The carrier will need to refer to the following sections:
  - i. Pushing the wheelchair, (Section 5.8).

  - iii. How to disconnect the batteries, (Section 6.7)
  - iv. How to disconnect the drive, (Section 5.8)
- Transporting the chair, (Section 7.0).
- Medium to long term storage, follow the instructions, (Section 8.5).

#### 7.4 General transport warnings

#### $\stackrel{!}{\square}$ warning!

- No changes or replacements must be made to the anchorage points/car fastenings on the wheelchair, or to constructional elements or parts of the frame without consulting Sunrise Medical.
- The wheelchair should be inspected by a Sunrise Medical authorised dealer/service agent, before reuse following involvement in any type of vehicle collision.

#### 8.0 Maintenance & cleaning

The wheelchair's lifespan is dependent on it being well maintained.

For information concerning specific settings, maintenance or repair work, please contact your authorized Sunrise dealer. Always be sure to mention the model, year of manufacture and identification number provided on the identification plate of the wheelchair when contacting your dealer.

# **CAUTION!**

The wheelchair should be serviced by your authorized Sunrise dealer once a year or, in the case of intensive use, every six months. For a list of approved authorised dealers in your area please contact Sunrise Medical Service Centre.

The contact details of your local Sunrise medical service centre can be found on the inside front cover of this booklet. National and International Website addresses are on the back cover.

#### 8.1 Maintenance



### **!**\ WARNING!

Loose fasteners should be re-tightened according to the installation instructions.

Please refer (unless otherwise specified) to the general table below for needed Torques.

Torque Setting Guide				
M4	3.0 Nm			
M5	5.9 Nm			
M6	10 Nm			
M8	25 Nm			
M10	48 Nm			
M12	84 Nm			

Note: it will be necessary to use a torque wrench.

- Chest straps should be replaced at the first indication of damage and/or excessive wear.
- If a broken or loose component is found, discontinue use immediately and contact your authorised Sunrise Medical supplier for replacement.
- Check all Velcro fastening straps for correct adhesion when pressed together.
- Ensure that any contamination, such as fluff, hair, etc is removed from the Velcro straps. Such contamination may affect adhesion.

### NARNING!

- If you are in any doubt about the performance requirements of your wheelchair contact your Sunrise Medical authorised dealer.
- After performing any maintenance or repairs on the wheelchair you must make sure that it is functioning correctly before it is used.
- All fasteners must be replaced like for like using the correct length, tensile strength and materials.
- When replacing self-locking nuts, or nuts/studs secured with a thread locking solution, ensure that a suitable thread locking solution is reapplied to the fastener.

#### **Daily checks**

Perform the daily check routine before driving as described in chapter 5.1

#### Weekly checks

Perform the weekly check routine before driving as described

#### **Checking Parking brake:**

This test should be carried out on a level floor with at least one meter clearance all around the chair.

- Switch on the control system.
- Check that the battery gauge remains on, or flashes slowly, after one second.
- Push the joystick slowly forwards until you hear the parking brakes operate.
- The chair may start to move.
- Immediately release the joystick. You must be able to hear each parking brake operate, (click), within a few seconds.
- Repeat the test a further 3 times, pushing the joystick slowly backwards, left and right.

#### Checking connectors & cables:

- Make sure that all connectors are securely mated.
- Check the condition of all cables and connectors for damage.

#### **Checking controller:**

- Check the thin rubber gaiter or boot around the base of the joystick shaft for damage or splitting. Check visually only, do not handle the gaiter.
- Make sure that all components of the control system are securely mounted. Do not over-tighten any securing screws.

#### **Checking controls:**

- Switch on the hand control Do the lights flash? This signifies that there is a fault in the electronic system. Refer to chapter 9 for basic troubleshooting
- Operate all of the electric options, including lights and indicators, (if fitted), to make sure that they work correctly.
- With the seating in an elevated position, drive the wheelchair to make sure that the 'creep' mode works which will slow the wheelchair.
- Drive the wheelchair in each of the drive profiles to make sure the wheelchair performs as it did before.

### ✓! WARNING!

- If you are in any doubt about the performance requirements of your wheelchair contact your Sunrise Medical authorised dealer.
- After performing any maintenance or repairs on the wheelchair you must make sure that it is functioning correctly before it is used.
- A complete inspection, safety check and service should be made by a Sunrise Medical authorised dealer at least once per year.
- All fasteners must be replaced like for like using the correct length, tensile strength and materials.
- When replacing self-locking nuts, or nuts/studs secured with a thread locking solution, ensure that a suitable thread locking solution is reapplied to the fastener.
- Check all Velcro fastening straps for correct adhesion when pressed together.
- Ensure that any contamination, such as fluff, hair, etc is removed from the Velcro straps. Such contamination may affect adhesion.

#### Monthly checks

Perform the monthly check routine before driving as described below.

- All fasteners should be checked monthly for wear, such as loose bolts or broken components.
- Check all straps monthly for fraying, ripped seams or other indications of excessive wear damage. Discontinue use if damage is found.

#### 8.2 Tyre maintenance and pressures

#### 8.2.1 Tyre pressure



# **⚠** CAUTION!

If pneumatic tyres are fitted to your wheelchair it is important to regularly check the air pressure and for signs of wear. The maximum tyre pressures are listed in Section 8.3, if in doubt check the markings on the side of the wheelchair tyre for guidance.

**NOTE:** It is important that drive wheels are inflated to equal pressures as a pair, and likewise the rear. The inflator pump provides the safest method of inflating your wheelchair tyres and the pressure can be checked with a standard motor vehicle pressure gauge.



# ⚠ DANGER!

- Do not inflate beyond the maximum allowed tyre pressure.
- · Always use the pump that is supplied with the chair,

#### 8.2.2 Tyre wear

When inspecting the tyres for signs of wear, look for significant scuff marks, cuts and a diminished tyre tread. Tyres will need to be changed when the tread cannot be seen over the complete surface of the tyre, (Fig.8.1).



#### 8.2.3 Drive wheel tyre repair

#### To remove the wheel/tyre:


Please see below and next page for photographs.


- · Remove the colour ring.
- Use a 16 mm AF lug nut to loosen the 3-Socket studs, (Fig. 8.2).
- Jack the base/battery box up and support it using blocks, (Fig. 8.3).
- Remove the 3 studs and pull the wheel away from the hub.
- Remove the valve cap and let the air out of the tyre by gently pressing the valve stem with a small screw driver, (Fig. 8.4).
- There are 6 rim studs that must be loosened/tightened in the number order shown, (Fig. 8.5).
- Use a 5.0 mm Allen key to release/tighten the studs. (Fig. 8.5).
- Lift the inner rim off the tyre wall (Fig. 8.6).
- Lift the tyre and tube off the outer rim, (Fig.8.7).
- Gently get a hold of the inner tube just behind the valve.
- Carefully feed the tube out of the tyre, (Fig.8.8).
- Ensure all parts are clean before reassembling, (Fig.8.9).


#### To refit


- Place the tube inside the tyre and rest the tyre on the outer rim.
- Align the valve stem with the cut out in the rim.
- Face the valve stem outward.
- Place the inner rim over the tyre, tube and outer rim.
- Align the cut out to fit over the valve stem and match it up to the cut out in the outer rim, (Fig.8.10).
- Make sure the stud mounts are in line on both rims.
- Tighten the studs in the sequence shown in Fig.8.5, taking care not to pinch the tube.
- Slowly inflate to the pressures quoted in section 8.1.1.
- Refit the wheel back onto the motor shaft and fix it safely with the 3 studs to a torque of 34Nm.


For solid tyres the sequence is the same, just discount the references to the inner tube and valve.

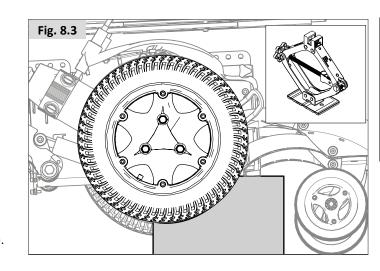


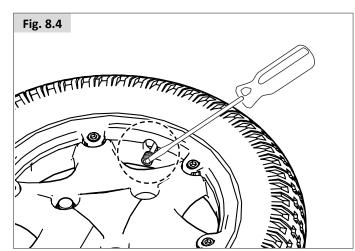


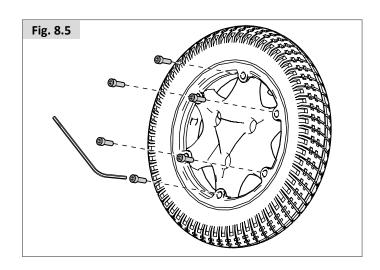


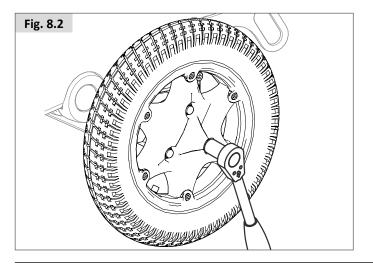

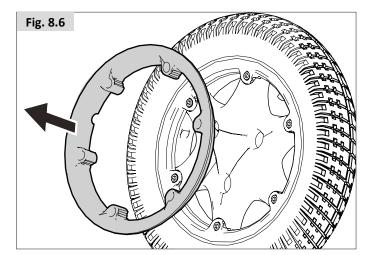

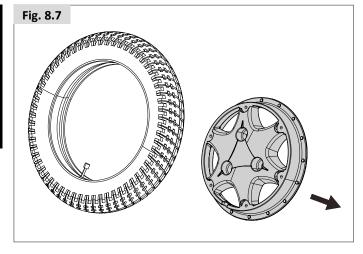

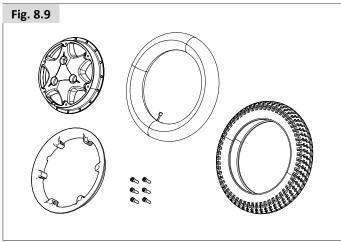

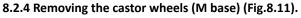

- Use a 17.0 mm spanner/socket to undo the axle bolt.
- Remove the nyloc nut.

### Removing the castor wheels (F/R base) (Fig.8.12)


- Use a 13.0 mm wrench and 5 mm Allen key to undo loosen the screws.
- Remove both screws


#### Refitting:


When refitting do not try to force the bolt through. Gently move the wheel back and forth until the bolt slips through then tighten to a torque of 14Nm.




Always use a new nyloc nut when refitting.

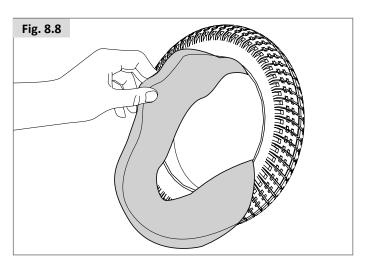

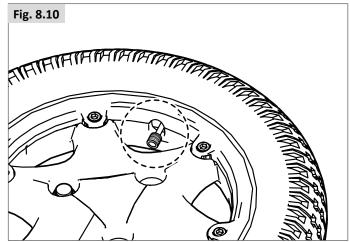

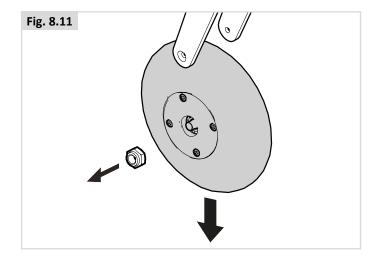

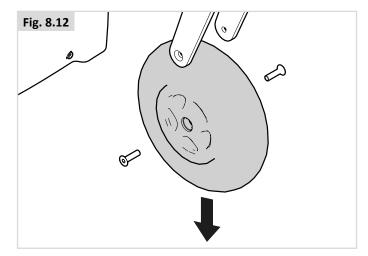

Maintenance & Inspection Schedule	Daily*	Weekly	Monthly	Annually
Daily check routine as described in chapter 5.1	*			
Weekly check routine as described in chapter 8.1		*		
Clean wheelchair upholstery 🕮 Seating system		*		
Complete inspection, safety check and service should be made by a Sunrise Medical authorised dealer.				*

#### 8.3 Wheels & tyres maintenance

To ensure that your wheelchair functions properly, it is essential that the tires are kept at the correct pressure. The correct tyre pressure is indicated on the side of the tyre.

Tyres that are too soft will have a negative effect on the wheelchair's performance. Moving the wheelchair will require more energy, making more demands on the batteries. Furthermore, wear on under inflated tyres is increased

When inspecting the tyres for signs of wear, look for significant scuff marks, cuts and a diminished tyre tread. Tyres will need to be changed when the tread cannot be seen over the complete surface of the tyre.

Castor wheel		Max. tyre pressure	
6"	Q500 M	Solid wheels	
9" (2.80/2.50-4)	Q500 F Q500 R	2.4 bar max.	35 PSI
10" (3.00-4)	Q500 F Q500 R	2.4 bar max.	35 PSI
Drive wheel		Max. tyre pressure	
13" (2.50-8)	Q500 M Q500 F Q500 R	2.4 bar max.	35 PSI max.
14" (3.00-8)	Q500 F Q500 R	3.5 bar max.	50 PSI max.



- Do not inflate beyond the maximum allowed tyre pressure.
- Never use a forecourt pump
- Before tyres are repaired they must first be fully deflated!

#### 8.4 Lighting maintenance:



### 🖺 DANGER!

Maintaining lights and indicators is a safety critical task. If your lights and indicator system shows a defect, please contact your Sunrise Medical Approved Dealer.

All lights and indicators are state of the art LED low energy, no maintenance units. There are no light bulbs used. The high intrinsic reliability of these units means that they are extremely unlikely to fail under normal usage. If a failure should occur, (may be as a result of an impact), the damaged light assembly must be replaced in its entirety. The individual LED's cannot be replaced.



### 🔼 CAUTION!

- We recommend the use of Sunrise Medical authorized spare parts only.
- Be advised that all lighting circuits are electronically protected. In the event of a short circuit current will be limited to a safe level. Once the fault is removed the system resets automatically.

#### 8.5 Cleaning and disinfection

The wheelchair should be wiped over once per week with a slightly damp, not wet, cloth and any fluff or dust that has accumulated around the motors should be blown or dusted away.



### **!**\ CAUTION!

Make sure that you dry all parts of your wheelchair if it becomes wet or damp after cleaning or if it is used in a wet or damp atmosphere.



### 🖺 DANGER!

It is important that should the wheelchair be used by more than one person it is cleaned thoroughly to ensure there is no cross infection.

#### Hygiene measures when being re-used:

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a suitable disinfectant for rapid alcoholbased disinfection of medical products and medical devices. Please take into account the manufacturer's instructions for the disinfectant you are using.



### **!**\ CAUTION!

- Do not use solvents, bleaches, abrasives, synthetic detergents, wax polishes or aerosols.
- Disinfectants may be used in dilution as specified by their manufacturer.
- Ensure surfaces are rinsed with clean water and dried thoroughly.



### ✓!\ warning!

- Always read the label on any commercial or domestic cleaning substances.
- Always follow the instructions carefully.

#### 

Should the control of your wheelchair become soiled or dirty, it can be wiped with a damp cloth with a dilute disinfectant until clean.



#### DANGER!

Important: If the wheelchair is to be used by more than one person, follow the cleaning and disinfection instructions carefully to ensure that there is no cross infection.

#### 8.6 Medium to long term storage:

When storing your wheelchair for long periods of time (in excess of one week), follow these simple instructions:

- Fully charge the wheelchair for at least 24 hours.
- Disconnect the charger.
- Disconnect the batteries.



### ✓ WARNING!

Never store your wheelchair:

- · Outside.
- In direct sunlight, (plastic parts may discolour).
- Near a source of direct heat.
- In a damp environment.
- In a cold environment.
- With the batteries/battery boxes connected, (even if the controller is switched off).

Avoiding all of the above will minimise battery deep cycle discharge and extend battery lifetime.

When returning the wheelchair to use, please reconnect the batteries/battery boxes and charge the wheelchair for at least 24 hours before use.

#### 9.0 Disposal

The symbols below mean that in accordance with local laws and regulations your product should be disposed of separately from household waste. When this product reaches the end of its life, take it to the local collection point designated by local authorities. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is recycled in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations and national requirements.









In the following section, there is a description of the materials used on the wheelchair, in view of the disposal or recycling of the wheelchair and its packaging.

There may also be special local regulations in force with regard to disposal or recycling, these must be taken into account when disposing of your wheelchair. (This can include the cleaning or decontamination of the wheelchair prior to disposal).

**Aluminium:** Castor forks, wheels, side-guards for the chassis.

Steel: Fixing points, quick-release axle

**Plastic:** Handles, tube stoppers, castors and wheel/tyre **Packaging:** Plastic bags made of soft polyethylene, cardboard

Disposal or recycling should be done through a licensed agent or authorised place of disposal. Alternatively your wheelchair may be returned to your dealer for disposal.



### 10.0 Trouble shooting

If the wheelchair is not working as it should, check the following points.

- Check whether the batteries are charged.
- Turn the wheelchair off and then back on again.
- Check whether the battery plugs are all securely in place.
- Check whether the freewheel lever is in the DRIVE position
- Check the position of the speed regulator.
- Make sure that the controller isn't locked 

   Controller

If the problem has not been resolved:

Controller

### 11.0 Technical specifications: Applicable norms / standards



This product complies with the regulations and guidelines for medical aids and carries a CE symbol. The product meets the requirements and standards below. These are checked by independent institutions.

Standard	Definition / description			
EU guideline 93/42 EEC	Applicable as mention	oned in Appendix 1		
EN 12182: 2012 Class B	Assistive products for requirements and te	or persons with disabi est methods	lity - General	
Class B	Q500 M	Q500 F	Q500 R	
EN 12184: 2014 Class B	Electrically powered - Requirements and	wheelchairs, scooter test methods	rs and their chargers	
Class B	Q500 M	Q500 F	Q500 R	
ISO 7176-8: 2014	Requirements and test methods for impact, static and fatigue strengths			
ISO 7176-9: 2009	Climate tests for electric wheelchairs			
ISO 7176-14: 2008	Requirements and test methods for control systems for electric wheelchairs			
ISO 7176-16: 2012	Requirements for resistance to ignition of upholstered parts			
ISO 7176-19: 2008	Wheelchairs - Part 19: Wheeled mobility devices for use as seats in motor vehicles: The Q-series meet the crash test requirements as described in ISO 7176-19			
	Q500 M	Q500 F	Q500 R	

Test Dummy Weight			
SEDEO Pro	SEDEO Lite		
160 kg	136 kg		
SEDEO Pro	SEDEO Lite		
160 kg	136 kg		
N/A	N/A		
SEDEO Pro	SEDEO Lite		
104 kg	75 kg		

Model	Q500	) series	
Type / configuration Mid wheel drive	Q500 M		
Maximum user weight			
In combination with Sedeo Lite	136 kg	300 lbs.	
In combination with Sedeo Pro	160 kg	350 lbs.	
EN12184 class:		В	

		Metric sys	Metric system values		Imperial system values & alternative	
Description		Minimum	Maximum	Minimum	Maximum	
Total length (including legrests)		IVIIIIIIIIIII	1 Waxiiiiaiii	Nimitani	IVIGAIITIGITI	
Mid wheel drive						
Q500 M Sedeo Lite	Central footplate	1100 mm	1130 mm	43,3"	51,2"	
Q500 M Sedeo Pro	Swingaway hangers	1280 mm	1280 mm	50,4"	50,4"	
Total width	0. 11 . 0.					
Q500 M	13" Drive wheels	610 mm	620 mm	24,0"	24,4"	
Total weight						
Basic configuration, r	o batteries					
Q500 M Sedeo Lite		130 kg	138 kg	286.6 lbs.	304.2 lbs.	
Q500 M Sedeo Pro		130 kg	138 kg	286.6 lbs.	304.2 lbs.	
Total weight influence	rS .					
Batteries						
50 Ah AGM (C20)	(Set of 2)		7 kg	+ 59.5		
60 Ah GEL (C20)	(Set of 2)		3 kg	+ 94.8		
80 Ah GEL (C20)	(Set of 2)		50.2 kg		lbs.	
Transport weight of th		24	24,2 kg		53,3 lbs.	
Dynamic stability: Max	•		20			
	wheel drive		8°	14,0 % 82,7"		
Min. braking distance a	t maximum speed Q500 M	2100 mm		82,7"		
Static stability Q500 M Sedeo Lite	Dannenda / Hansanda / Cidan	ays 9°/9°/9°	1 420/450/420	15 00//15 00//15 00/	1 220/ /2/0/ /220/	
Q500 M Sedeo Lite	Downwards / Upwards / Sidew Downwards / Upwards / Sidew		13°/15°/13° 13°/15°/13°	15,8%/15,8%/15,8% 15,8%/15,8%/15,8%		
Range of action	Downwards / Opwards / Sidew	ays 9/9/9	13 /15 /15	15,8%/15,8%/15,8%	23%/20%/23%	
I =						
	have a negative influence on the range of action driving on slopes, exposure to temperatures bel					
, , , ,	it use of powered seat options.	,				
6 kph	80 Ah batteries Q500 M	32.5 km	40.0 km	20.2 miles	24.9 miles	
8 kph	80 Ah batteries Q500 M	30.0 km	37.5 km	18.6 miles	23.3 miles	
10 kph	80 Ah batteries Q500 M	27.5 km	35.0 km	17.1 miles	21.7 miles	
12.5 kph	80 Ah batteries Q500 M	25.0 km	32.5 km	15.5 miles	20.2 miles	
Climbing capability for						
Q500 M	Mid wheel drive	75 mm	100 mm	2.95"	3.94"	
Max speed forward						
Q500 M	Mid wheel drive	6 kph	12.5 kph	4 mph	7.7 mph	
Turning radius						
Q500 M	Mid wheel drive	560 mm	630 mm	22.0"	24.8"	
Turning space / reversi						
Q500 M	Mid wheel drive	1100 mm		43.0"		
Ground clearance						
Q500 M	Mid wheel drive		90 mm		3.5"	

	Metric sys	Metric system values		Imperial system values & alternative	
Description	Minimum	Minimum Maximum		Maximum	
Maximum battery dimensions (I x b x h)		260 x 172 x 210 mm		10.2 x 6.8 x 8.3"	
Battery capacity	50 Ah / 60	50 Ah / 60 Ah / 80 Ah		50 Ah / 60 Ah / 80 Ah	
Maximum permissible charging voltage		24V		24V	
Maximum charging current		12 A (rms)		12 A (rms)	
Connector type	Controller manual		Controller manual		
Insulation	Class 2 double insulated		Class 2 doub	ole insulated	

Model	Q500	series		
Type / configuration Front wheel drive	Q5	Q500 F		
Maximum user weight				
In combination with Sedeo Lite	136 kg	300 lbs.		
In combination with Sedeo Pro	160 kg	350 lbs.		
EN12184 class:		В		

			Metric system values		Imperial system values & alternative	
Description			Minimum	Maximum	Minimum	Maximum
Total length (including	legrests)					
Front wheel drive						
Q500 F Sedeo Lite	Central footplate	e	1280 mm	1340 mm	50.4"	52.8"
Q500 F Sedeo Pro	Central footplat	e	1280 mm	1340 mm	50.4"	52.8"
Total width						
Q500 F	13" Drive wheel	S	615 mm	615 mm	24.2"	24.2"
Q500 F	14" Drive wheel	S	640 mm	640 mm	25.2"	25.2"
Total weight						
Basic configuration, i	no batteries					
Q500 F Sedeo Lite			130 kg	135 kg	286.6 lbs.	297.6 lbs.
Q500 F Sedeo Pro	+ anti Shear		133 kg	138 kg	293 lbs.	304 lbs.
Total weight influence	rs					
Batteries						
50 Ah AGM (C20)	(Set of 2)		27	′ kg	+ 59.5	lbs.
60 Ah GEL (C20)	(Set of 2)		43	kg kg	+ 94.8	lbs.
80 Ah GEL (C20)	(Set of 2)		48.4 kg		+ 106.7 lbs.	
Transport weight of th	e heaviest part		24.	2 kg	53.4 lbs.	
Dynamic stability: Max	•					
Q500 F Froi	nt wheel drive			8° 14.05 %		5 %
Min. braking distance a	at maximum speed	Q500 F	2900 mm		114.17"	
Static stability						
Q500 F Sedeo Lite	Downwards / Up		9°/9°/9°	15°/15°/ 12°	15.8%/15.8%/15.8%	26% / 26% / 21.2%
Q500 F Sedeo Pro	Downwards / Up	wards / Sideways	9°/9°/9°	15°/15°/ 12°	15.8%/15.8%/15.8%	26% / 26% / 21.2%
Range of action						
Obstacles, rugged terrain,	have a negative influence on the driving on slopes, exposure to t nt use of powered seat options.					
6 kph	80 Ah batteries	Q500 F	32.5 km	40.0 km	20.2 miles	24.9 miles
8 kph	80 Ah batteries	Q500 F	30.0 km	37.5 km	18.6 miles	23.3 miles
10 kph	80 Ah batteries	Q500 F	27.5 km	35.0 km	17.1 miles	21.7 miles
12.5 kph	80 Ah batteries	Q500 F	25.0 km	32.5 km	15.5 miles	20.2 miles
Climbing capability for	obstacles					
Q500 F	Front wheel drive		70 mm		2.75"	
Max speed forward						
Q500 F	Front wheel drive		6 kph	12.5 kph	4 mph	7.7 mph
Turning radius						
Q500 F	Front wheel drive		735 mm	735 mm	28.9"	28.9"
Turning space / reversing width						
Q500 F	Front wheel drive		1240 mm		48.8"	
Ground clearance						
Q500 F	Front wheel drive			60 mm		2.36"

	Metric sys	tem values	Imperial system values & alternative		
Description	Minimum Maximum		Minimum	Maximum	
Maximum battery dimensions (I x b x h)	260 x 172 x 210 mm			10.2 x 6.8 x 8.3"	
Battery capacity	50 Ah / 60 Ah / 80 Ah		50 Ah / 60 Ah / 80 Ah		
Maximum permissible charging voltage	24V			24V	
Maximum charging current		12 A (rms)	! A (rms)		
Connector type	Controller manual		Controller manual		
Insulation	Class 2 double insulated		Class 2 double insulated		

Model	Q500 s	series		
Type / configuration Rear wheel drive	Q500 R			
Maximum user weight				
In combination with Sedeo Lite	136 kg	300 lbs.		
In combination with Sedeo Pro	160 kg	350 lbs.		
FN12184 class:	B			

			Metric system values		Imperial system values & alternative	
Description		Minimum	Maximum	Minimum	Maximum	
Total length (including l	egrests)					
Rear wheel drive						
Q500 R Sedeo Lite	Central footpl	ate	1290 mm	1370 mm	50.8"	53.9"
Q500 R Sedeo Pro	Swingaway ha	ngers	1110 mm	1180 mm	43.7"	46.4"
Total width						
Q500 F	13" Drive whe	els	615 mm	615 mm	24.2"	24.2"
Q500 R	14" Drive whe	els	640 mm	640 mm	25.2"	25.2"
Total weight						
Basic configuration, no	b batteries					
Q500 R Sedeo Lite			130 kg	135 kg	286.6 lbs.	297.6 lbs.
Q500 R Sedeo Pro +	- anti Shear		133 kg	138 kg	293 lbs.	304 lbs.
Total weight influencers	3					
Batteries						
50 Ah AGM (C20)	(Set of 2)		27	kg kg	+ 59.5	lbs.
60 Ah GEL (C20)	(Set of 2)		43	kg kg	+ 94.8	lbs.
80 Ah GEL (C20)	(Set of 2)		48.4 kg		+ 106.7 lbs.	
Transport weight of the	heaviest part		24.	2 kg	53.4 lbs.	
Dynamic stability: Maxi						
<del></del>	wheel drive		10°		17.6 %	
Min. braking distance at	maximum speed	Q500 R	2100 mm		82.7"	
Static stability						
Q500 R Sedeo Lite		Jpwards / Sideways	9°/9°/9°	13°/15°/ 13°	15.8%/26%/15.8%	23% / 26% / 23%
Q500 R Sedeo Pro	Downwards / L	Jpwards / Sideways	9°/9°/9°	13°/15°/ 13°	15.8%/26%/15.8%	23% / 26% / 23%
Range of action						
NB: The following aspects h						
Obstacles, rugged terrain, a						
freezing point and frequent			22.5.1	1 40.01	20.2 '!	240 :1
6 kph	80 Ah batteries	Q500 R	32.5 km	40.0 km	20.2 miles	24.9 miles
8 kph	80 Ah batteries	Q500 R	30.0 km	37.5 km	18.6 miles	23.3 miles
10 kph	80 Ah batteries	Q500 R	27.5 km	35.0 km	17.1 miles	21.7 miles
12.5 kph	80 Ah batteries	Q500 R	25.0 km	32.5 km	15.5 miles	20.2 miles
Climbing capability for o	obstacies	Q500 R	00	I	2.14"	
Rear wheel drive Climbing capability with			80 mm		3.14"	
, , ,	i kercilmber for obstac		100	l mana	3.9	,,,,
Rear wheel drive Q500 R Max speed forward		100	mm I	3.5		
Q500 R	Rear wheel drive		6 kph	12.5 kph	4 mph	7.7 mph
Turning radius	hear writer urive		ОКРП	12.5 KPII	4 IIIpii	7.7 IIIpii
Q500 R	Rear wheel drive		950 mm	950 mm	37.4"	37.4"
Turning space / reversir			730 111111	730 111111	57.4	37.4
Q500 R	Rear wheel drive		1258 mm		50.6"	
Ground clearance	THE WITCH WITCH		1233 111111		30.0	
Q500 R	Rear wheel drive			65 mm		2.55"

	Metric system values		Imperial system values & alternative		
Description	Minimum Maximum		Minimum	Maximum	
Maximum battery dimensions (I x b x h)	260 x 172 x 210 mm			10.2 x 6.8 x 8.3"	
Battery capacity	50 Ah / 60 Ah / 80 Ah		50 Ah / 60 Ah / 80 Ah		
Maximum permissible charging voltage	24V			24V	
Maximum charging current		12 A (rms)		12 A (rms)	
Connector type	Controller manual		Controller manual		
Insulation	Class 2 double insulated		Class 2 double insulated		

Technical specifications. SEDEO Lite seating sy	Jacon					
		Metric system values		Imperial system values & alternative		
		Minimum	Maximum	Minimum	Maximum	
Seat angle adjustment						
Powered tilt with COG adjustment optio	nal	0° -	30°	-5° -	-5° - 25°	
Powered tilt with COG adjustment optio	nal	-5° -	- 25°	0° - 30°		
Powered tilt with COG adjustment optio	nal	0° -	50°	0° -	0° - 50°	
Effective seat depth		410 mm	560 mm	16.1"	22.0"	
Effective seat width		385 mm	510 mm	15.2"	20.1"	
Seat to floor height						
Filler Module (No powered tilt in space, no	seat elevator)	,				
pos. 1 60 Ah batteries	0° seat angle	415	mm	16	.3"	
pos. 2 60 or 80 Ah batteries	0° seat angle		mm		.3"	
pos. 3 60 or 80 Ah batteries	0° seat angle	465	mm	18	.3"	
pos. 4 60 or 80 Ah batteries	0° seat angle	490	mm	19	.3"	
Powered tilt & seat elevator (300 mm)						
pos. 1 60 or 80 Ah batteries	0° seat angle	440 mm	740 mm	17.3"	29.1"	
pos. 2 60 or 80 Ah batteries	0° seat angle	465 mm	765 mm	18.3"	30.1"	
pos. 3 60 or 80 Ah batteries	0° seat angle	490 mm	790 mm	19.3"	31.1"	
Backrest angle						
Tool angle adjustable back		86°,90°,94°,98°,102°,106°		86°,90°,94°,98°,102°,106°		
Powered recline		85° -	120°	85° - 120°		
Backrest height (seatplate to top of back cushi	on)	460 mm	510 mm	18.1"	20.1"	
Flip back armrest		,				
Armrest pad height (seatplate to top armpa	d)	190 mm	320 mm	7.5"	12.6"	
Armrest pad depth (on armrest profile)		100	100 mm		9"	
Armrest pad angle (vertical)		0° -	15°	0° -	15°	
Single Post Flip back armrest						
Armrest pad height (seatplate to top armpa	d)	250 mm	400 mm	9.8"	15.7"	
Armrest width (on seatframe)		-20 mm	+20mm	-0,79"	+0,79"	
Armrest pad width (on armrest profile)		-20	mm	-0,	79"	
Armrest depth (on seatframe)		250	mm	9,	8"	
Armrest pad depth (on armrest profile)		140 mm		5,5"		
Armrest pad angle (vertical)		0° -	15°	0° -	15°	
Center Mount Legrest		,				
Lower leg length (from seatplate)		305 mm	460 mm	12"	18.1"	
Knee angle		70° -	120°	70° -	120°	
Ankle angle		+20°40°		+20°40°		
Power Center Mount Legrest						
Lower leg length (from seatplate)		360 mm	540 mm	14.2"	21.2"	
Knee angle			180°	90° -		
Max length compensation		260 mm		10.2"		
Ankle angle		+35°	15°	+35°	15°	
Swing away SEDEO LITE legrests			·		1	
Lower leg length (from seatplate)		350 mm	510 mm	13.8"	20.1"	
Knee angle			0°		0°	
Ankle angle		0° -	180°	0° -	180°	

	Metric system values			tem values & native
	Minimum	Maximum	Minimum	Maximum
Swing away SEDEO LITE legrests, manual elevating / power elevating				
Lower leg length (from seatplate)	350 mm	510 mm	13.8"	20.1"
Knee angle	0° -	70°	0° -	70°
Ankle angle	0° -	180°	0° -	180°
Swing away UNI legrests				
Lower leg length (from seatplate)	350 mm	510 mm	13.8"	20.1"
Knee angle	7(	0°	70	0°
Ankle angle	0° -	180°	0° -	180°
Swing away SEDEO PRO legrests				
Lower leg length (from seatplate)	350 mm	510 mm	13.8"	20.1"
Knee angle	70°		7(	o°
Ankle angle	0°	180°	0° -	180°
Swing away SEDEO PRO legrests, manual elevating / power elevating				
Lower leg length (from seatplate)	350 mm	510 mm	13.8"	20.1"
Knee angle	0° -	70°	0° - 70°	
Ankle angle	0° -	180°	0° - 180°	
SEDEO Multi-Position headrest mount				
Headrest height adjustment range	170	mm	6.	7"
Headrest depth adjustment range	200	mm	7.	9"
Headrest angle (vertical)	-45° -	- +45°	-45° -	+45°
Headrest width	-20 mm	20 mm	-0.8"	+0.8"
Lateral Supports				
Lateral support width adjustment range	90 mm		3.	5"
Lateral support height adjustment range	170 mm		6.	7"
Lateral support depth adjustment range	45 mm		1.8"	
Pommel				
Pommel height (from seatplate to top of cushion)	170 mm	250 mm	6.7"	9.8"
Pommel depth adjustment range	65 ו	mm	2	5"

Technical specifications: SEDEO Pro seating system		Metric sys	tem values	Imperial syst alterr		
		Minimum	Minimum Maximum		Minimum Maximum	
Seat angle adjustment						
Powered tilt with COG adjustment optional			· 30°	0° - 30°		
Powered tilt with COG adjustment	optional		- 25°	-5° - 25°		
Powered tilt with COG adjustment	optional		· 50°	0° - 50° -5° - 45°		
Powered tilt with COG adjustment	optional		- 45°			
Effective seat depth		400 mm	600 mm	15.7"	23.6"	
Effective seat width		380 mm	560 mm	15"	22.0"	
Seat to floor height Filler Module (No powered tilt in space, no seat elevat	orl					
pos. 1 60 or 80 Ah batteries	0° seat angle	/15	mm	16.	ວ"	
pos. 2 60 or 80 Ah batteries	0° seat angle		mm	17.		
pos. 3 60 or 80 Ah batteries	0° seat angle		mm	18.		
pos. 4 60 or 80 Ah batteries	0° seat angle		mm	19.		
Powered tilt & seat elevator (300 mm)	o scat angle	+30		13.	.5	
pos. 1 60 or 80 Ah batteries	0° seat angle	415 mm	715 mm	16.3"	28.1"	
pos. 2 60 or 80 Ah batteries	0° seat angle	440 mm	740 mm	17.3"	29.1"	
pos. 3 60 or 80 Ah batteries	0° seat angle	465 mm	765 mm	18.3"	30.1"	
pos. 4 60 or 80 Ah batteries	0° seat angle	490 mm	790 mm	19.3"	31.1"	
Backrest angle		122				
Tool angle adjustable back		90° -	· 120°	90° -	120°	
Powered recline			· 130°	90° -		
Powered recline with anti shear			· 150°	90° - 150°		
Backrest height (seatplate to top of back cushion)		530 mm	720 mm	20.9"	28.3"	
Flip back armrest / Single post removable armrest						
Armrest pad height (seatplate to top armpad)		260 mm	380 mm	10.2"	15"	
Armrest width (on seatframe)		-20 mm	+20 mm	-0.8"	+0.8"	
Armrest pad width (on armrest profile)		-20	mm	-0.	8"	
Armrest depth (on seatframe with short seat rail)		250 mm		10.8"		
Armrest pad depth (on armrest profile)		100 mm		3.9"		
Armrest pad angle (vertical)		0° - 15°		0° - 15°		
Reclining Flip back armrest						
Armrest pad height (seatplate to top armpad)		250 mm	400 mm	9.8"	15.7"	
Armrest pad width (on armrest profile)			mm	-0.		
Armrest pad depth (on armrest profile)			mm	5		
Armrest pad angle (vertical)		0° -	· 15°	0° -	15°	
Center Mount Legrest				"		
Lower leg length (from seatplate)		305 mm	460 mm	12"	18.1"	
Knee angle			· 120°	70° -		
Ankle angle		+20*	40°	+20°	40*	
Power Center Mount Legrest Lower leg length (from seatplate)		360 mm	540 mm	14.2"	21.2"	
Knee angle			· 180°	90° -		
Max length compensation			mm	10.		
Ankle angle			15°	+35°15°		
Swing away legrests		1,33		133		
Lower leg length (from seatplate)		350 mm	510 mm	13.8"	20.1"	
Knee angle			0°	7(		
Ankle angle			180°	0° - 1		
Swing away legrests, manual elevating / power elevating		L				
Lower leg length (from seatplate)		350 mm	510 mm	13.8"	20.1"	
Knee angle		0° -	- 70°	0° -	70°	
Ankle angle		0° -	180°	0° - 1	180°	
SEDEO Multi-Position headrest mount						
Headrest height adjustment range			mm	6.		
Headrest depth adjustment range			200 mm		9"	
Headrest angle (vertical)			- +45°	-45° -		
Headrest width		-20 mm	20 mm	-0.8"	+0.8"	
Lateral Supports						
Lateral support width adjustment range		90 mm		3.5"		
Lateral support height adjustment range		170 mm		6.7"		
Lateral support depth adjustment range		45	mm	1.5	8"	
Pommel						
Pommel height (from seatplate to top of cushion)		170 mm	250mm	6.7"	9.8"	
Pommel depth adjustment range			mm	2		
Pushbar height adjustment range		140	mm	5.	0	


#### 12.0 Warranty

#### THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical\* provides a guarantee, as set out in the warranty conditions, for products to its customers covering the following.

#### Warranty conditions:

- Should a part or parts of the product require repair or replacement as a result of a manufacturing and/or material fault within 24
  months, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing
  defects
- 2. To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty. Should you be using the product outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer The product must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
- 3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the product in accordance with point 1.
- 4. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.
- 5. Claims from this warranty shall not arise, if a repair or replacement of a product or a part is required for the following reasons:
  - a. Normal wear and tear, which include but is not limited to the following parts where fitted; batteries, armrest pads, upholstery, tyres, brakes shoes, ferrules, etc.
  - b. Any overloading of the product, please check the EC label for maximum user weight.
  - c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
  - d. Accessories have been used which are not specified as original accessories.
  - e. The product or part having been damaged by neglect, accident or improper use.
  - f. Changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
  - g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.
- 6. This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical"
- \* Means the Sunrise Medical facility from which the product was purchased.

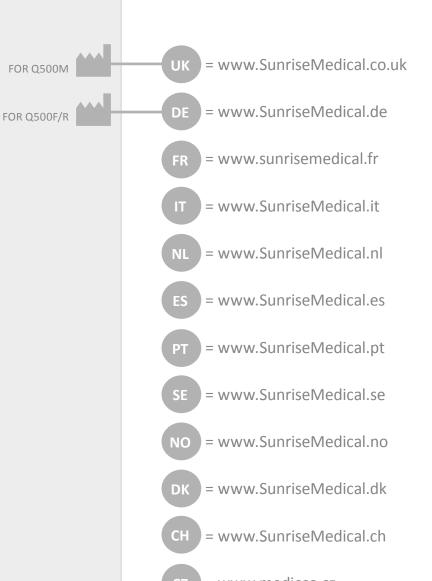

- cz = www.medicco.cz
- **PL** = www.Sunrise-Medical.pl
- **EU** = www.SunriseMedical.eu
- AU = www.sunrisemedical.com.au
- us = www.SunriseMedical.com







OM\_Q500 M-F-R\_EU\_EN\_FR\_NL\_Rev.B\_2018-09-06